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0234A-16TA

ZONE-QUICK

ZONE-QUICK is a special cotton thread (manufactured by Yokota Co., Ltd.) treated with a pH indicator phenol red (phenolsulfonphthalein). It was developed by Dr. Hikaru Hamano, et al., Department of Ophthalmology, Osaka University Medical School in 1982.

ZONE-QUICK is yellow (acidic) in color and is ensured for its stability as well as its water absorption qualities. When the thread comes in contact with tears, it changes to a light red color (basic).

The results yielded with ZONE-QUICK are considered to reflect the amount of tears in the conjunctival sac. It has been found clinically useful in screening patients whose tear volumes are rather small. It has also been found to be particularly effective in evaluating the possibilities of wearing contact lenses and help to predict those individuals who may have problems with certain types of contact lenses.

[Characteristics]

- (1) Easy to handle
- (2) Rapid testing (15 seconds for each eye)
- (3) No discomfort or pain as with the Schirmer tear test

[Prior to testing]

- (1) The test requires no anesthesia.
- (2) If eyedrops have been used, the test should be performed at least five minutes later.
- (3) This test may also be performed during contact lenses wear.

[Instructions for removing the test thread from the package]

- (1) Two sterilized threads are contained in each aluminum package.
- (2) Take them out by gently peeling the plastic film covering from the unsealed end of the aluminum sheet.

[Test procedure]

The eyes are tested one at a time by the following procedure:

- (1) The folded 3mm end of the thread should be bent open at an angle that allows easy placement onto the palpebral conjunctiva with forceps (Fig. 1).
- (2) As in the Schirmer test, the thread is placed at a point approximately 1/3 of the distance from the lateral canthus of the lower eyelid with the eye in the primary position.
- (3) The lower eyelid is pulled down slightly, and the folded 3mm portion of the thread is placed on the palpebral conjuncta at the position specified in (2) (Fig. 2).

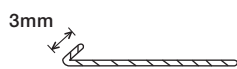
Fig. 1. Angle at the folded 3 mm portion.

Fig. 2. Point of placement of the thread.

- (4) Each eye is tested with the eyes open for 15 seconds.
During the test the patient is instructed to look straight ahead and blink normally.
- (5) When 15 seconds have elapsed, the lower eyelid is gently pulled down, and the thread gently removed with an upward motion. Care should be taken to pull the lower eyelid down before removal to avoid any patient discomfort.
- (6) The entire length of the red portion of the thread is measured in millimeters from the very tip regardless of the fold.
- (7) A watch may be used for the test; however, an electronic metronome has been found most convenient to measure the time.

[Criteria]

Wet lengths approximately 10mm or less are considered small.

[Warnings]

This device should be used only by physicians, ophthalmologists, optometrists, opticians or technicians trained in the technique of this test.

[Precautions]

- (1) Since tear volume is liable to vary, repeating the test on different days gives a more reliable result.
- (2) This test on rare occasion may induce a slight mechanical irritation leading to lacrimation in some patients. In such cases, it may be necessary to use other tear tests in addition to this test.
- (3) This test is intended for single patient use only. Contents are sterile if package is unopened and undamaged. Do not use if package has been previously opened.

[Storage]

Store in dark place at room temperature

[Expiry date]

3 years

* Expiry date is printed on the side of box.

[Package]

50 Aluminum package sets (100 Threads) per box.

[Reference]

- 1) Hikaru Hamano et al.: Contact Wear and Tear Secretion Test.
J. Jpn. C. L. Soc. 24(3): 287-291, 1982
- 2) Tamotsu Hamano: The Clinical Significance of The Phenol Red Thread Tear Test. Folia Ophthalmol. Jpn. 42(5): 719-727, 1991
- 3) Takashi Hamano: Dry Eye and the Contact Lenses. Journal of the Eye. 4(3): 341-347, 1987.
- 4) Chihiro Kobayashi: Dry Eye and the Contact Lens. Journal of the Eye. 8(7): 1043-1049, 1991.
- 5) Tamotsu Hamano et al.: Tear Volume in Relation to Contact Lens Wear and Age.
The CLAO Journal 16(1): 57-61, 1990.
- 6) Randall Sakamoto et al.: The Phenol Red Thread Tear Test: A Cross-Cultural Study.
Invest. Ophthalmol. Vis. Sci. 34(13): 3510-3514, 1993.
- 7) Simon A. Little and Adrian S. Bruce: Repeatability of the phenol-red thread and tear thinning time tests for tear film function. Clinical & Experimental Optometry 77(2): 64-68, 1994.

Manufacturer

AYUMI Pharmaceutical Corporation

12-15, Ginza 4-chome, Chuo-ku, Tokyo 104-0061 Japan